


Confirm the Study is on the Cloud-PACS

1. Once the study has finished uploading, log in to the exam platform to review the study.


The **[My Studies]** lists all uploaded studies with patient demographics.

Note the patient's name, ID, DOB.

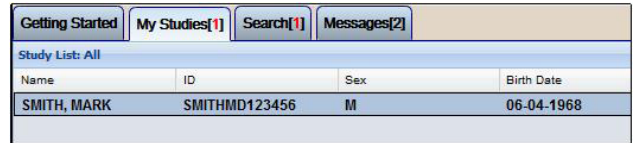

Name	ID	Sex	Birth Date
SMITH, MARK	SMITHMD123456	M	06-04-1968

2. You can double-click the study to open the study image viewer and, after status shows **[Reported,]** the final report, which can be opened/downloaded as a PDF with the **[Show PDF]** option.

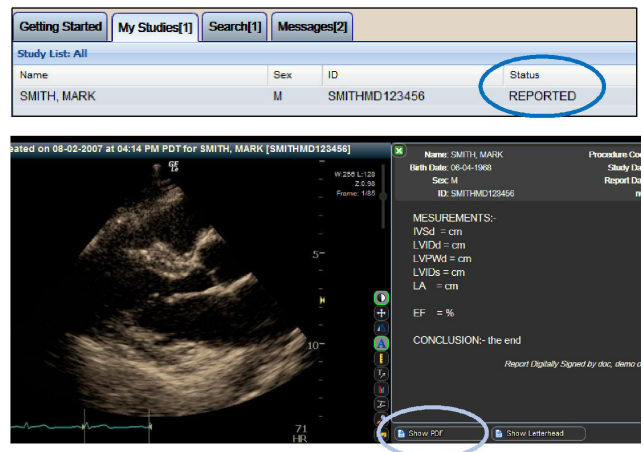

The top screenshot shows the 'My Studies' table with the following data:

Name	Sex	ID	Status
SMITH, MARK	M	SMITHMD123456	REPORTED

The bottom screenshot shows the study image viewer with the following details:

dated on 08-02-2007 at 04:14 PM PDT for SMITH, MARK (SMITHMD123456)

Name: SMITH, MARK
Birth Date: 06-04-1968
Sex: M
ID: SMITHMD123456

MEASUREMENTS:-
IVSd = cm
LVPWd = cm
LVPWd = cm
LVIDs = cm
LA = cm
EF = %

CONCLUSION:- the end

Report Digitally Signed by dsc, demo

Show PDF Show Letterhead

3. The next step, if you need a specialist to interpret the exam, is to contact CompuMed with a service request. Please see the **[CompuMed Reference Sheet]**.

If you have any questions, please call 310-258-5012 or support@compumedinc.com

www.compumedinc.com